

WARRANTY CERTIFICATE

The manufacturer's warranty has a duration of 24 (twenty-four) months. The warranty covers replacement or free repair of the defective part as confirmed by the dealer or by a third party appointed by the dealer.

The warranty becomes null and void if the diffuser is not properly used according to the instructions of the manufacturer. It is equally null and void should the diffuser be modified, repaired or disassembled outside the premises and control of the dealer.

SALE DATE -----

SELLER STAMP AND SIGNATURE-----

NEVER REPLACE THE CAPSULE BEFORE COMPLETION OF THE CYCLE
DO NOT REMOVE THE ADHESIVE LABEL UNDERNEATH UNIT

TECHNICAL FEATURES:

Power supply: 220-240 V

Network frequency: 50 Hz

Power consumption: max 15 W

The diffuser respects the European safety standards
PATENTED SYSTEM



www.kontak.it

PROPOL

therapy



ENVIRONMENT PROGRAM


Environmental sanitation completely natural with the spread of the volatile fraction of propolis. Scientific studies shows the reduction of airborne microbes up to 71, 3% in only three days of use (Biologi Italiani, 1996).




AEROSOL MASK PROGRAM

A healthy, effective and easy way to take care your well-being, just breathing! 100% natural prevention.

PROPOL therapy

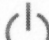
is equipped with multi-function keyboard that allows you to customize its use. Place the diffuser on a flat surface, plug it in and press 

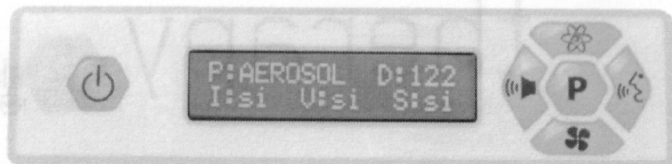

Select the desired program:

P ROOM

To get the best results in the prevention of respiratory diseases and for a full environmental sanitation, KONTAK recommends the use of the diffuser 24 hours/7 days a week. ONLY in this program it is possible to enable or disable the following functions:

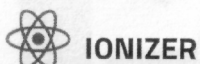

IONIZER

The ionizer enhances the sanitizing properties of the propolis. Through the process of ionization, the molecules of propolis released into the air, take on negative electrical charge and attract airborne particles of opposite polarity, such as bacteria, smoke, viruses and dust. The air is thus free of harmful substances and pollutants, and the respiratory system can fully enjoy the active ingredients of propolis.. Disabling allows the use of the diffuser with children under 3 years of age.



FAN

The diffuser without the fan option provides coverage up to 30 sq m. Activation of the fan can double the field of action in order to saturate environments up to 60 square meters.



SOUND

Disabling the sound eliminates the beep that alerts user of the end of the operating cycle of the capsule.


P AEROSOL MASK

After selecting AEROSOL MASK program, wait for the time required for reaching the ideal temperature of melting the Propolis (5 min). It is recommended that sessions are a minimum of 15 minutes. The kit supplied includes 1 pediatric mask, 1 adult mask and 1 connection tube.

Each capsule is disposable and has a operating life of approximately 122 hours. The display shows the countdown of the hours of usage (D). The end of the cycle of the capsule is indicated by a beep and by the following indication on the display:

**REPLACE CAPSULE
RESET CYCLE**

At the end of the duration of the capsule it is **NECESSARY TO RESET** the operating system. The process is the following:

1. Turn on the diffuser
2. Press the button  for about 15 seconds (you first hear a continuous beep and then an alternating beep).
3. Remove finger from button and replace the consumed capsule with a new capsule to start a new cycle

When the cycle is completed, it will be visible inside the capsule, a residue of propolis depleted, now devoid of active principles.

PROPOL uses exclusively capsules from KONTAK.
therapy



Display information is available in 5 languages: Italian, English, German, French and Spanish.